



Fitting Eyezen[™]+ Lenses

Eyezen+ lenses require the following standard measurements:

- Monocular Distance PD
- Fitting Height: Simply measure from center pupil to the lowest point of the lens to acquire fitting height. Recommended minimum fitting height is **15mm**.

Eyezen+ lenses with embedded Smart Blue Filter[™] feature are also recommended with Crizal[®] No-Glare treatment for the most comprehensive eye protection.

The following laser engravings located on the nasal side of each lens will identify the **Eyezen+** design received.

Eyezen+ 0



Eyezen+ 1




Eyezen+ 2



Eyezen+ 3



Visit EyezenPro.com/resources for additional information and materials.

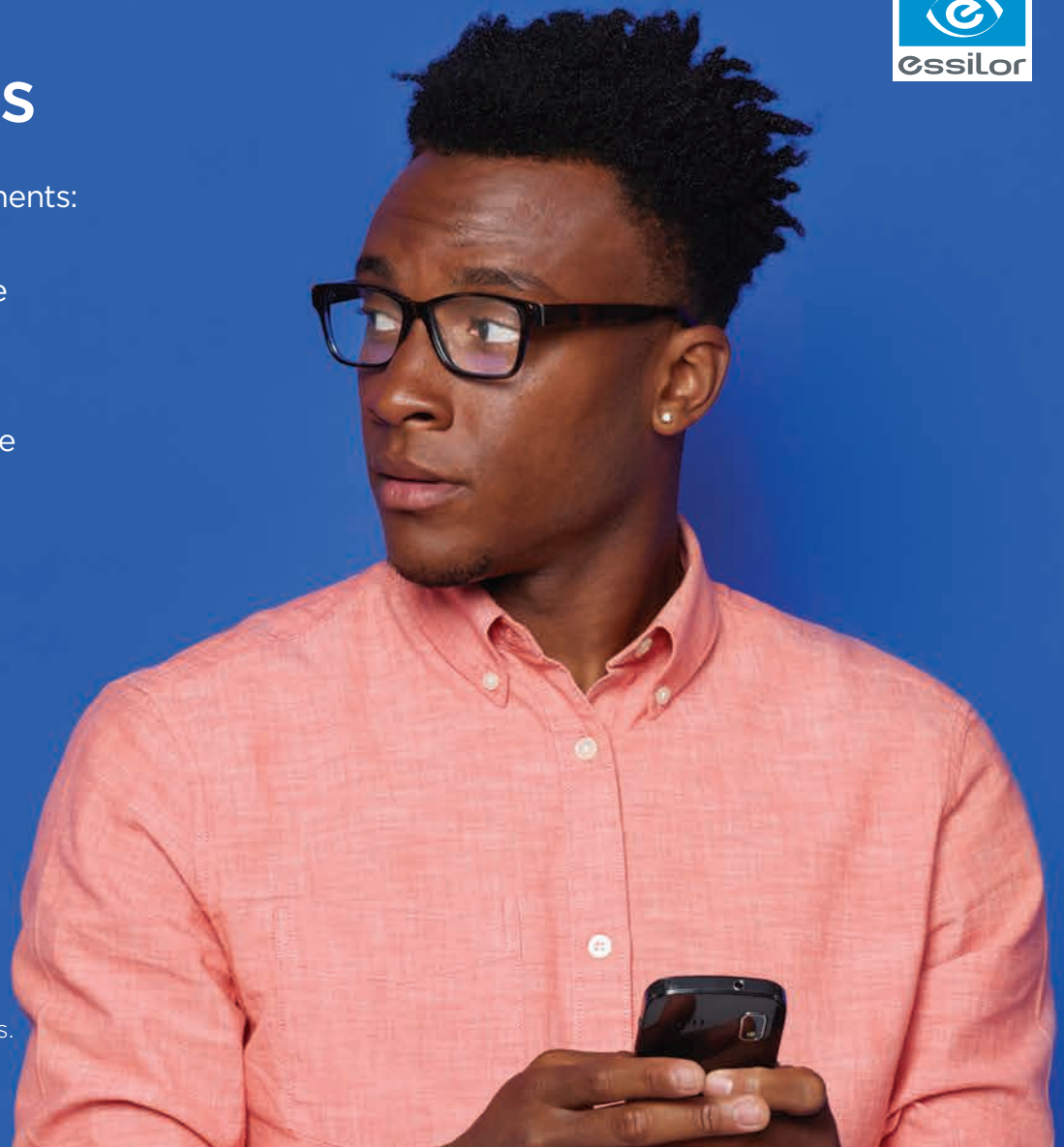

Embedded **Harmful Blue Light*** Protection.

*Eyezen+ lenses block at least 20% of Harmful Blue Light, which are the high energy waves found between 415 and 455nm (Blue-Violet light) and are suspected to be the most toxic to retinal cells.

Eyezen[™]+

EYEZEN™+ LENSES AVAILABILITY

for Eyecare Professionals

Effective JULY 2017

MATERIAL	PRODUCT	INDEX	COLOR	POWER RANGE	BASE CURVES	MAX CYLINDER
PLASTIC	1.50 Index Clear	1.50		-10.00 to +6.00	2.00, 3.50, 4.25, 6.25, 7.25	-6.00
	1.50 Index Essilor Polarized	1.50	Gray, Brown, Gray-Green, Gray-1, Brown-1	-10.00 to +6.00	2.00, 3.25, 4.25, 6.25, 8.25	-4.00
	1.50 Index Xperio UV™ Polarized	1.50	Gray, Brown, Gray-Green, Gray-1, Brown-1	-10.00 to +6.00	2.00, 3.25, 4.25, 6.25, 8.25	-4.00
	1.50 Index Transitions® Signature® VII	1.50	Gray, Brown, Graphite Green	-10.00 to +6.00	2.00, 3.50, 4.25, 6.25, 7.25	-6.00
	1.50 Index Transitions® Vantage®	1.50	Gray	-10.00 to +6.00	2.25, 4.25, 5.25, 8.25	-6.00
	1.50 Index Transitions® XTRActive®	1.50	Gray	-10.00 to +6.00	2.25, 4.25, 5.25, 8.25	-6.00
	1.50 Index Transitions® XTRActive®	1.50	Brown	-10.00 to +6.00	2.00, 3.50, 4.25, 6.25, 7.25	-6.00
AIRWEAR® (POLYCARBONATE)	Clear	1.59		-10.00 to +8.00	1.50, 3.25, 4.25, 5.50, 7.00, 8.50	-6.00
	Essilor Polarized	1.59	Gray, Brown, Gray-Green, Gray-1, Brown-1, Copper, Ruby	-10.00 to +8.00	1.50, 3.25, 4.25, 5.50, 7.00, 8.50	-6.00
	Xperio UV™ Polarized	1.59	Gray, Brown, Gray-Green, Gray-1, Brown-1, Copper, Ruby	-10.00 to +8.00	1.50, 3.25, 4.25, 5.50, 7.00, 8.50	-6.00
	Transitions® Signature® VII	1.59	Gray, Brown, Graphite Green	-10.00 to +8.00	1.50, 3.25, 4.25, 5.50, 7.00, 8.50	-6.00
	Transitions® XTRActive®	1.59	Gray, Brown	-10.00 to +8.00	1.50, 3.25, 4.25, 5.50, 7.00, 8.50	-6.00
	Airwear® Transitions® Vantage®	1.59	Gray	-10.00 to +8.00	1.50, 3.25, 4.25, 5.50, 7.00, 8.50	-6.00
	Airwear Colors™	1.59	Gray, Brown, Gray-Green	-10.00 to +8.00	2.25, 4.25, 6.50, 7.50, 8.50	-6.00
TREXA™ (TRIVEX®)	Clear	1.53		-10.00 to +6.00	2.00, 3.50, 4.25, 6.25, 7.25	-6.00
	Transitions® Signature® VII	1.53	Gray, Brown	-10.00 to +6.00	2.00, 3.50, 4.25, 6.25, 7.25	-6.00
THIN & LITE®	1.60 Clear	1.60		-12.00 to +8.00	1.75, 2.75, 3.75, 6.00, 6.75, 8.50	-6.00
	1.60 Transitions® Signature® VII	1.60	Gray, Brown	-12.00 to +8.00	1.75, 2.75, 3.75, 6.00, 6.75, 8.50	-6.00
	1.67 Clear	1.67		-14.00 to +9.00	1.75, 2.75, 3.75, 6.00, 6.75, 8.00	-6.00
	1.67 Essilor Polarized	1.67	Gray, Brown, Gray-Green	-14.00 to +9.00	1.75, 2.75, 3.75, 5.25, 6.75, 8.00	-6.00
	1.67 Xperio UV™ Polarized	1.67	Gray, Brown, Gray-Green	-14.00 to +9.00	1.75, 2.75, 3.75, 5.25, 6.75, 8.00	-6.00
	1.67 Transitions® Signature® VII	1.67	Gray, Brown, Graphite Green	-14.00 to +9.00	1.75, 2.75, 3.75, 6.00, 6.75, 8.00	-6.00
	1.67 Transitions® XTRActive®	1.67	Gray, Brown	-14.00 to +9.00	1.75, 2.75, 3.75, 6.00, 6.75, 8.00	-6.00
	1.74 Clear*	1.74		-20.00 to +13.00	0.75, 1.75, 2.75, 3.75, 5.25, 6.50, 8.00, 10.00	-6.00
	1.74 Transitions® Signature® VII*	1.74	Gray	-20.00 to +13.00	0.75, 1.75, 2.75, 3.75, 5.25, 6.50, 8.00, 10.00	-6.00

NOTES: *Thin & Lite 1.74 lenses come systematic with choice of Crizal® Previncia, Crizal Sapphire UV™ Crizal Avancé UV™ or Crizal Alizé UV™
Please include fitting height when ordering Eyezen+ lenses. Recommended minimum fitting height is 15mm.

EYEZEN+ LENS DESIGNS

There are four unique **Eyezen+** designs. Although the recommended age ranges for each design provided below are based on research, as always, the Eyecare Professional is free to use discretion in prescribing **Eyezen+** enhanced single vision lenses.

Eyezen+ 0	patients age 17 and under
Eyezen+ 1	patients age 18 to 34
Eyezen+ 2	patients age 35 to 44
Eyezen+ 3	patients age 45 to 50**

**It is recommended that you consider progressive lenses for your patients age 45-50.

For more information on **Eyezen+** lenses, please visit EyezenPro.com

©2017 Essilor of America, Inc. All Rights Reserved. Unless indicated otherwise, all trademarks are the property of Essilor International and/or its subsidiaries in the United States and in other countries. Transitions, the swirl, Transitions Signature, and XTRActive are registered trademarks of Transitions Optical, Inc., used under license by Transitions Optical Limited. Photochromic performance and polarization are influenced by temperature, UV exposure, and lens material. Trivex is a registered trademark of PPG Industries Ohio, Inc. LEZN000063 SHK/ECSO 6/17